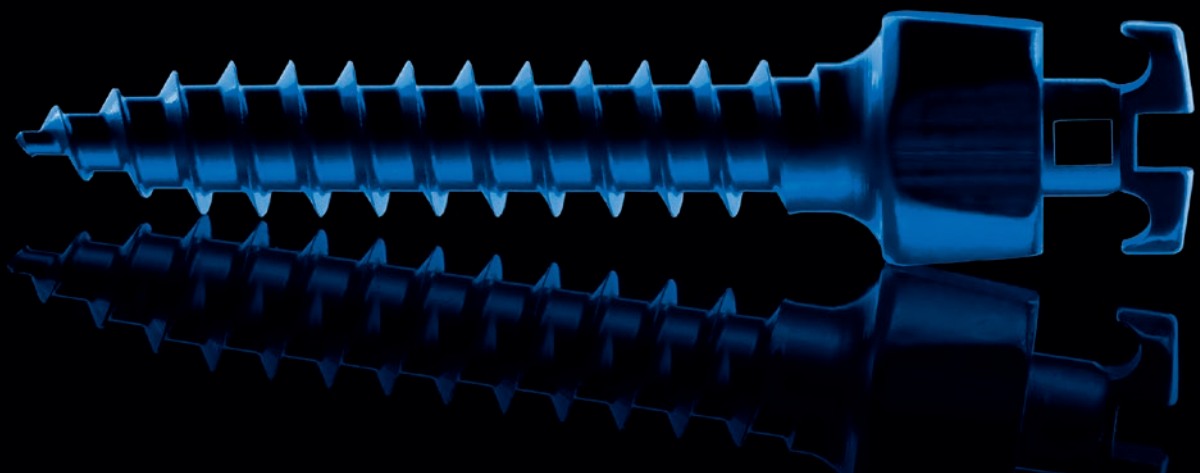
**psm25**  
1993-2018 | YEARS

**quattro**<sup>®</sup> .KFO

**New  
Neu**

**www.psm.ms** – implants for professionals

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2001

2002

Inventing the  
**quattro hook**  
screw family.

2003

2004

2005

Revolutionizing  
orthodontic  
anchorage with  
**quattro plus**

PRODUCTS  
PRODUKT LÖSUNG

quattro Video

BENEFit Video

**Orthodontic Mini-Anchor System**

The use of Mini-Anchor Screws or Temporary Anchorage Devices (TAD's) has become a standard of care within the orthodontic community over the past few years.

First offered in 2000 the **quattro** TAD design has continuously been changed and improved to become the **quattro** TAD family of today. The **quattro** TAD's even after more than a decade of production are still unmatched in terms of versatility, quality and maximal success rate. With the **quattro mini** screw a new even smaller bracket head design is launched in 2012 offering a new dimension in patient comfort based on the outstanding **quattro** design.

Since year 2000 close to 7 0.000 **quattro** TAD's have been placed in more than 50 countries worldwide.

The **quattro** System is ideally enhanced with the unique **BENEFit/BENEplate** System (developed in cooperation with University of Düsseldorf, Germany) offering exchangeable abutments and connecting plates. This opens up new possibilities in palatal based appliances popularly used for molar distalization (see QR Video).

**quattro** and **BENEFit** a complete TAD System using the same basic instrumentation keeping inventory manageable and very cost effective.

**Features:**

- Clinically Proven Applications
- Titanium Implant Material
- Self-Drilling & Self-Tapping Technique
- Reliable & Secure-Fixed Anchorage
- Innovative Head Attachment Design
- Stronger Conical Body Design
- Deeper Threads & Denser Pitch Design
- Various Screwhead Design
- Available in multiple Sizes

**Benefits:**

- Provides improved higher strength
- Immediate load-bearing of heavy forces
- Easy to combine with applications of orthodontic accessories
- Better mechanical retention
- Avoids gingival impingement
- Reduced treatment time
- Easy to insert and remove

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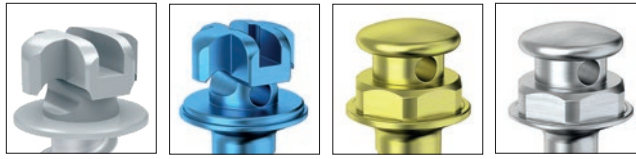
2006    2007/08    2009    2011    2012    2017

Perfecting TAD treatment with **benefit/beneplates**

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
### One Screw – Unlimited Possibilities

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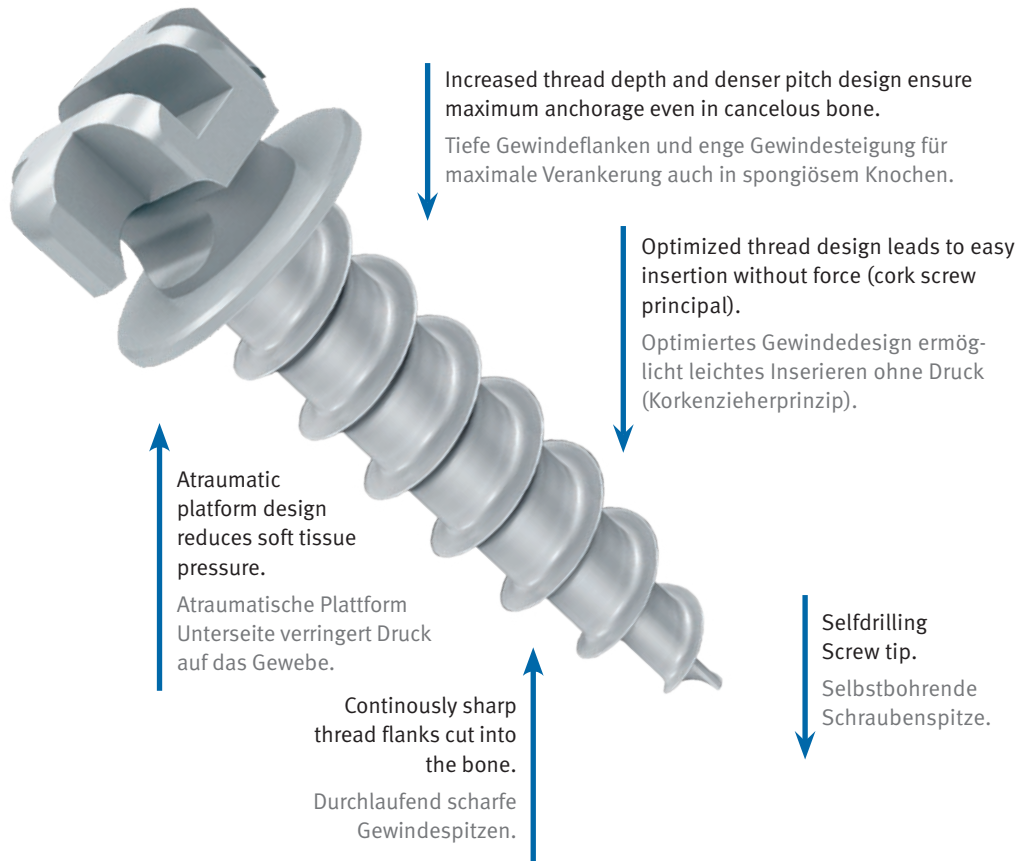DENTAL

Based on the successful **quattro** Screw the **quattro mini** Screw has been reduced in size in the head and neck area to meet compatible up with different anatomical conditions.

The design of the selfdrilling thread is identical to the PSM **quattro** and **BENEFit** Screws which have demonstrated outstanding primary stability and very low failure rates for many years. The extremely sharp threads and the optimized thread pitch facilitate the insertion tremendously and make the **quattro mini** an ideal addition to your selection of Standard Mini Anchor Screws.

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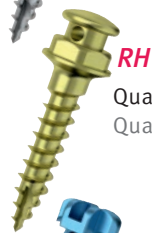
ST ■

■ Note:

All implants are supplied in a pre-sterilized single package (see page 13).

**RH:** Round head design  
**BH:** Bracket head design

\* Pre-drilling necessary  
\* Vorbohrung erforderlich

12-12609 DENTAL

Drill, 0.9x22 mm, WL 8.5 mm, blue, for 1.3 mm screws  
Bohrer, 0,9x22 mm, AT 8,5 mm, blau, für 1,3 mm Schrauben

11-12208 DENTAL

Drill, 1.2x22 mm, WL 8.5 mm, green for 1.6 mm screws  
Bohrer, 1,2x22 mm, AT 8,5 mm, grün für 1,6 mm Schrauben

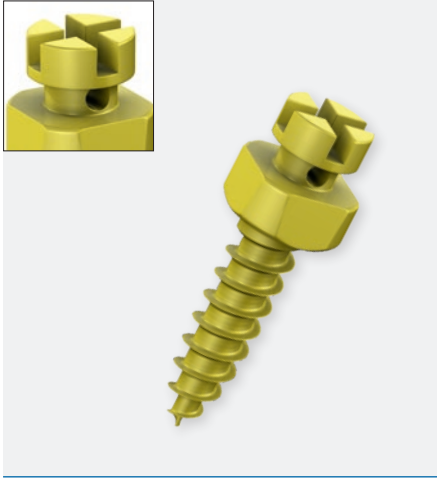
*quattro standard*

Packaging: 1 ea.  
VPE: 1Stk.

Material Titanium  
Material Titan

quattro Video

**quattro standard**

**Size (d x length)**

Abmessung (d x length)

				<b>ST</b>
<b>1.5</b>	<a href="#">ST-33-11507</a>	1.5 x 7 mm		
	<a href="#">ST-33-11509</a>	1.5 x 9 mm		
	<a href="#">ST-33-11511</a>	1.5 x 11 mm		
	<a href="#">ST-33-11513</a>	1.5 x 13 mm		
<b>2.0</b>	<a href="#">ST-33-12207</a>	2.0 x 7 mm		
	<a href="#">ST-33-12209</a>	2.0 x 9 mm		
	<a href="#">ST-33-12211</a>	2.0 x 11 mm		
	<a href="#">ST-33-12213</a>	2.0 x 13 mm		
	Emergency Screw Notfallschraube			
<a href="#">ST-33-12309</a>	2.3 x 9 mm			

*quattro hook*

Packaging: 1 ea.  
VPE: 1Stk.

**quattro hook**

**Size (d x length)**

Abmessung (d x length)

				<b>ST</b>
<b>1.5</b>	<a href="#">ST-33-21507</a>	1.5 x 7 mm		
	<a href="#">ST-33-21509</a>	1.5 x 9 mm		
	<a href="#">ST-33-21511</a>	1.5 x 11 mm		
<b>2.0</b>	<a href="#">ST-33-22007</a>	2.0 x 7 mm		
	<a href="#">ST-33-22009</a>	2.0 x 9 mm		
	<a href="#">ST-33-22011</a>	2.0 x 11 mm		
	<a href="#">ST-33-22013*</a>	2.0 x 13 mm		
	Emergency Screw Notfallschraube			
<a href="#">ST-33-22109</a>	2.3 x 9 mm			

**ST**

**Note:**

All implants are supplied in a pre-sterilized single package (see page 11).

\* as long as stock lasts  
\* Auslaufartikel

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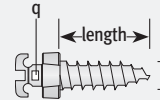
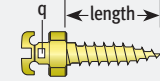
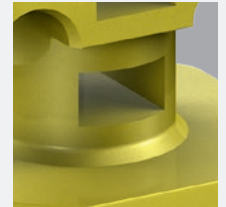
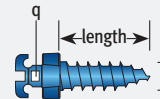
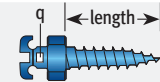
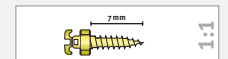
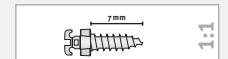
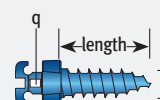
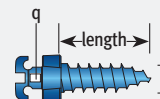
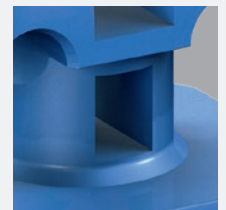
■ **Note:**

All trays are also suitable for use with **BENEFit** and **Mentoplates** implants and instruments.

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73-31960

**QB**

**quattro** large sterilization tray for instruments and implants, without content, with lid

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33-11500

**quattro / BENEFit Instrument Set**  
**consisting of:**

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33-18232

Screwdriver for contra-angled handpiece, 23 mm

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73-31970

QB

**quattro** small sterilization tray for instruments, without content, with lid

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33-11555

**quattro** Basic Instrument Set consisting of:

**Sterilization and Storage**

73-31970

**quattro** instruments tray

**Instruments**

10-61000

Screwdriver handle

33-18230

Screwholder, long, for 10-61000

**quattro** Basis Instrumenten Set bestehend aus:

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**quattro** Instrumenten-Behälter

**Instrumente**

Schraubendreher Handgriff

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INSTRUMENTS  
INSTRUMENTE

■ **Note:**

All instruments marked with the symbol **QB** can be used with **quattro** and **BENEFit** screws!

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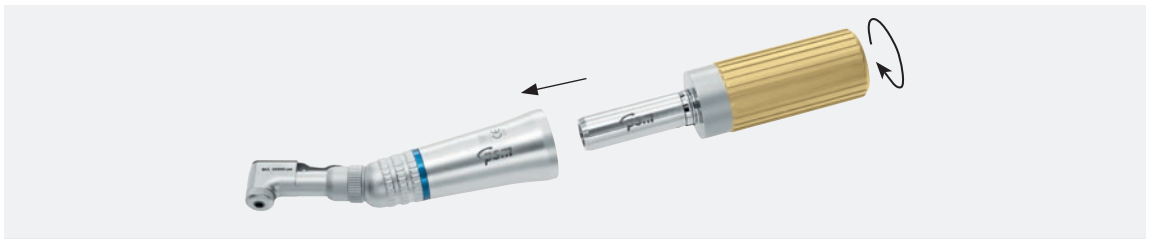
**33-18230** 66 mm **QB**

Screwholder, long, for **10-61000**  
Schraubenhalter, lang, für **10-61000**

The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use with an angled screwdriver.

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**10-63025** **QB**

Manually turned unit for contra-angled handpieces

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**33-54704** **QB**

Manually turned unit mod. to Pauls, with adjustable torque from 0 – 40 Ncm

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INSTRUMENTS  
INSTRUMENTE

All **quattro** screwholders are marked with an arrow indicating the slot, hole or hook position on the **quattro** screw during insertion.

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The screwholders with DENTAL attachment shaft can be used for a contra-angled handpiece and the thumb screwdriver (33-18236).

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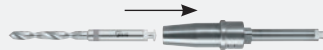
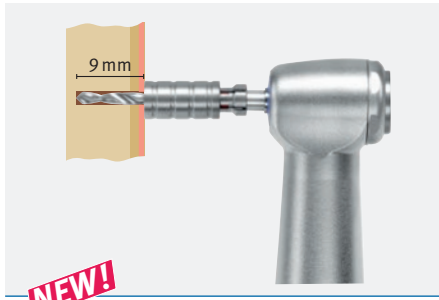
Video

The drill stop prevents the clinician from unintentionally drilling too deep. They assure an accurate working length from 7 to 13 mm in 2 mm increments. By sliding the stop sleeve over the appropriate PSM drill\*, the working length is adjusted to the desired depth. Removal of the sleeve is easy using the buffer.

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**Cordless Battery Screwdriver** with Torque Calibration System for the fixation of **quattro** and **BENEFit** Miniscrews.

The iSD 900 will guarantee precise torque control due to the unique – easy to use – torque calibration system (10–40 Ncm). Time-saving and precise, risk-free insertion and tightening of screws and abutments.

The complete Set contains a reliable and user friendly handpiece for Dental attachments. Reverse mode is indicated with sound. Standard rechargeable AAA Ni-Mh batteries can be used.

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## iSD900

BATTERY SCREWDRIVER  
AKKU SCHRAUBENDREHER

### FEATURES VORTEILE

- 10–40 Ncm torque in 1 or 5 increments
- 5, 10, 15, 25 rpm turning speed
- Reverse mode with sound
- Calibration facility
- Standard rechargeable batteries

### 33-900-00 iSD900 consisting of / bestehend aus:

33-900-04	Motor	Handgriff mit Motor
33-900-08	Handpiece DENTAL	Winkelansatz DENTAL
33-900-10	Quick Charger Unit	Schnellladestation
33-900-12	Power Cord	Netzkabel
33-900-14	Torque Calibrator	Drehmoment Kalibrierer
33-900-16	Calibration Burr	Kalibrierstift
33-900-18	On/Off Switch Lever	Ein/Aus Komfort-Bügel
33-900-20	Ni-Mh Rechargeable Battery (Pack of 2)	Ni-Mh Akkus (2 Stk.)

Torque Calibrator  
Drehmoment Kalibrierer

**STERILE PACKAGING**  
**STERILE VERPACKUNG**

Information

**Implants**  
**in single sterile packaging**

The outer and inner packaging contains labels indicating article number, LOT number, expiration date, etc. for the patient folder and for full traceability.

The sterile screws are contained in a small color-coded screw carrier which protects the screw and at the same time acts as a pick up tray. This carrier is placed in a clear, sealed blister package. The screw can easily be removed from the carrier by means of the self retaining screwdriver.

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KAT-030

**COURSES**  
**KURSE**

**psm** MEDICAL SOLUTIONS offers courses and webinars in various languages around the world.

Please check the currently available courses on our website <http://www.psm.ms/1/courses/>

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Web www.psm.ms

© 2018 **psm** MEDICAL SOLUTIONS | Germany

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of **psm** MEDICAL SOLUTIONS. Utilization and copies by third parties have to be authorized by **psm** MEDICAL SOLUTIONS. All rights reserved!

All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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